

Nepplus(NE-SM1) Cleaning info & Cautions



STEP 1
Mesh
cleaning




Dispose of residual medication after use and nebulize for 60 sec after filling with clean water.

Caution

- Never use a disinfectant solution including hypochlorite solution.

STEP 2
Accessories
cleaning

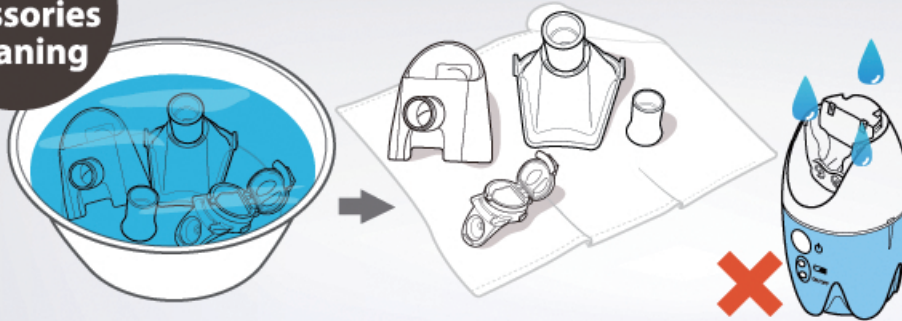

Gently shake parts in clean water and dry completely.

Caution

- Never use a disinfectant solution including hypochlorite solution.
- O-ring of the medication bottle could be missing when shake parts strongly.
- Never dry in a microwave oven and Baby bottle sterilizer
- Keep main case away from water

Disinfecting
(Once a week
at least)



Disinfect the medication bottle, mesh cartridge, mask and mask adapter by boiling for **1 minute** and dry completely.

Caution

- Do not boil over 1 minute
- Boil nebulizer parts before initial use
- Do not touch mesh with tweezers or objects

Filling with
medication



After filling with medication, use it promptly. (If the unit is not used within **10 minutes**, medication may leak)

Caution

- When not in use instantly, dispose of medication and keep the unit

Battery
storage



When not in use for long periods, charge battery and then separate it from unit. Keep unit dry.

Caution

- Check proper battery polarity when inserting one.